AED Tool

Installation and Use Guide

mindray

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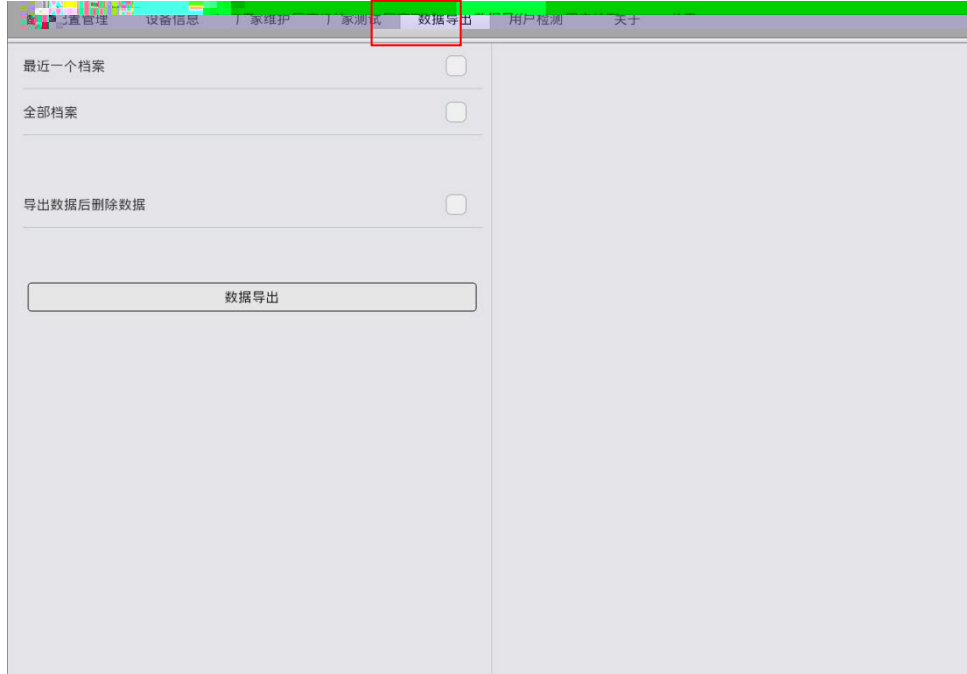
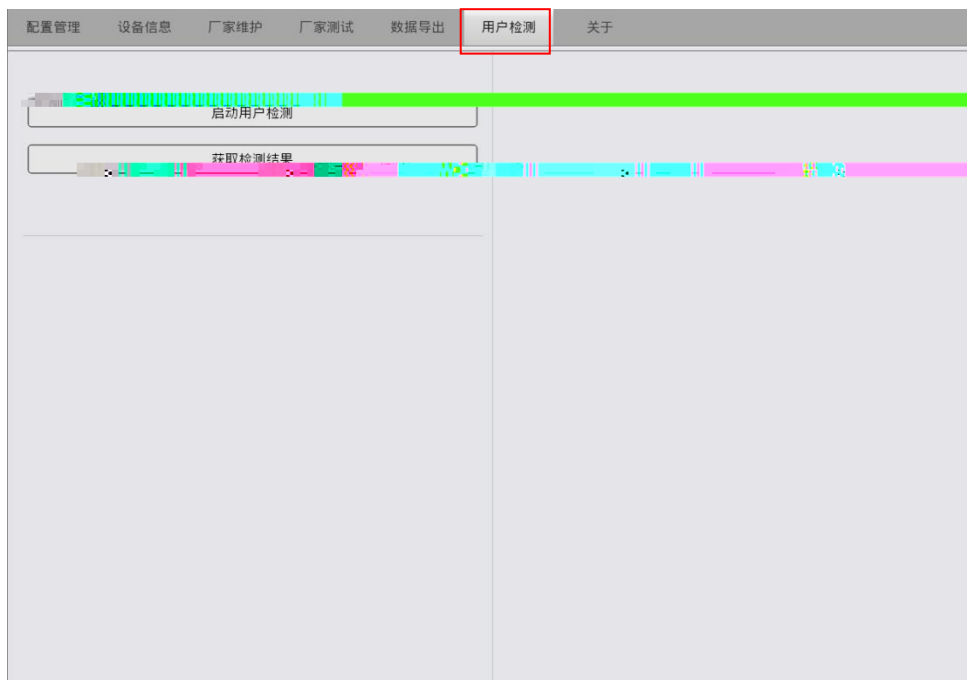
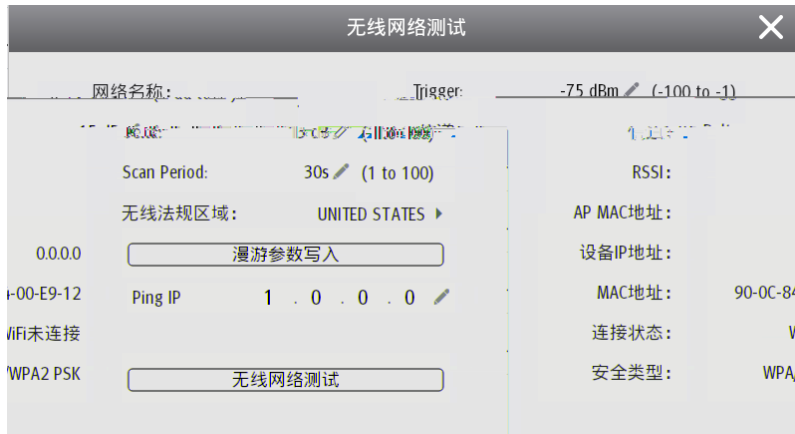
1.3.8

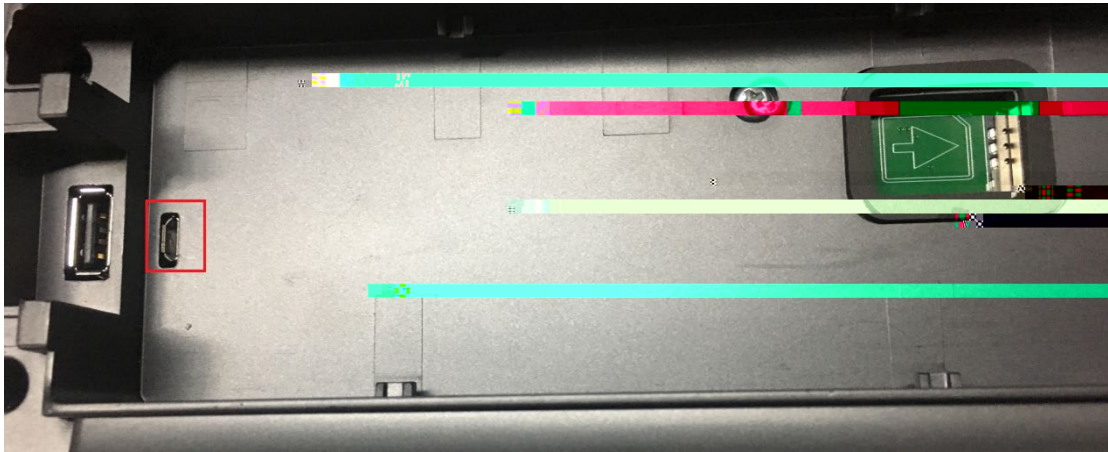
AED Tool

© Copyright 2019-2021 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

For this Operator's Manual, the issue date is 2021-06.

1.1 Installation Environment

1. The operating environment of AED tools is compatible with Windows 10 and Windows 7 Professional SP1 systems.
2. Connection method: support the device to connect the AED tools through the USB data cable; the red box in the figure below is the micro USB port of the device:


1.2 Software Installation

1.2.1 Installing the Software


1. Double click **AedToolInstaller.exe** and install the software.

Note: For 32-bit operating systems, first install the VCOM driver, double-click **lpc_driver_setup.exe** to install, and then install the configuration tool software **AedToolInstaller.exe**.

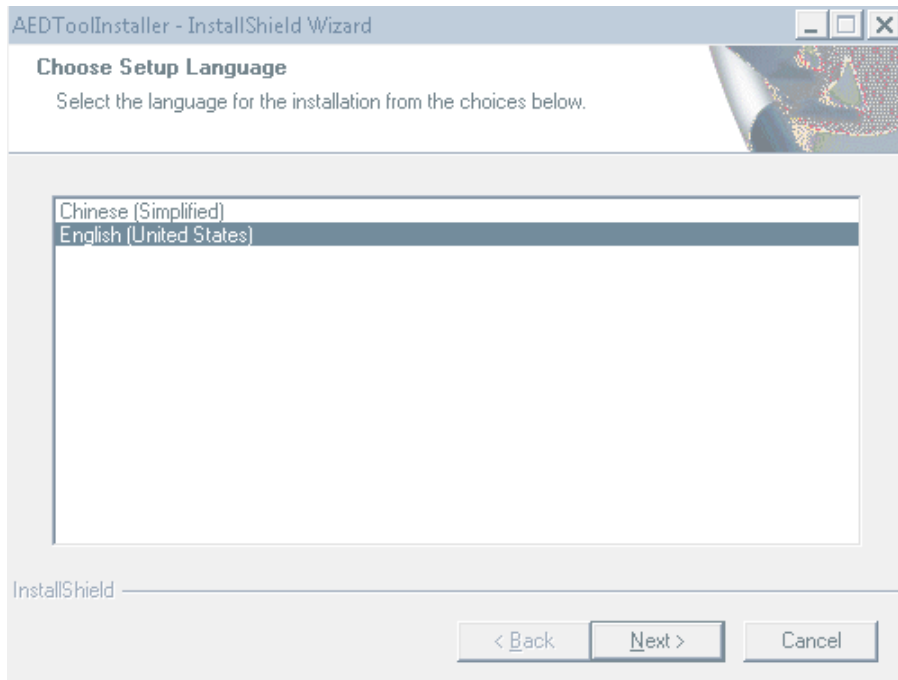2. Choose the setup language.

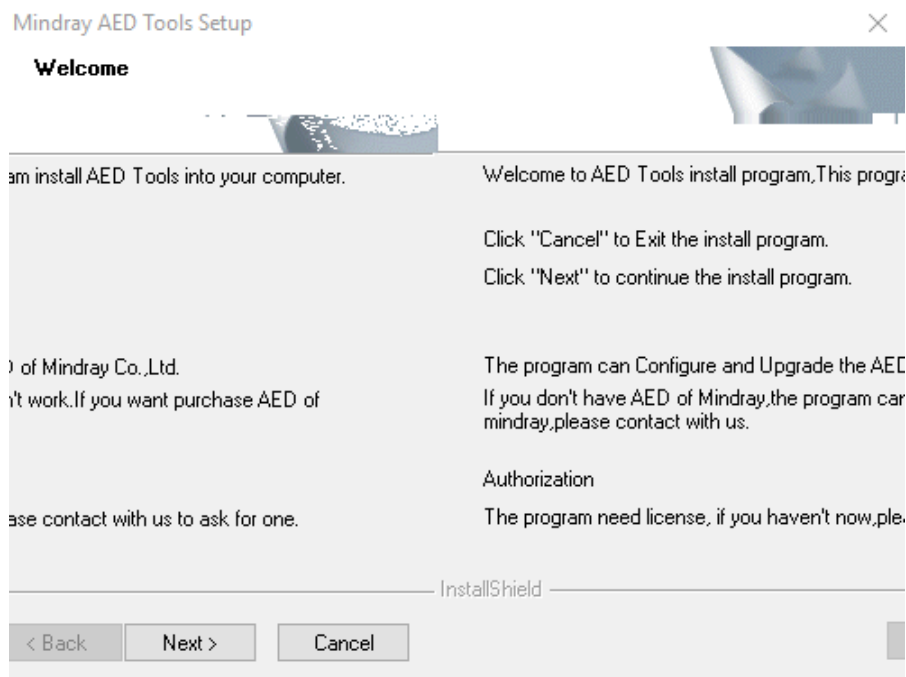3. Click Next, and be ready to install.

4. Click **Next**, and enter **Customer Information** and enter the serial number.

If you enter the administrator serial number, you may have administrator permissions: configuration management, device information, factory maintain, factory test, data export, and user test.

If you enter the common user serial number, you may have common user permissions: configuration management, device information, data export, and user test.

Mindray AED Tools Setup

Customer Information

Please enter your information.

Please enter your name, the name of the company for which you work and the product serial number.

User Name:

Company Name:

Serial Number:

InstallShield

< Back Next > Cancel

5. Click **Next**, and set a login password for the configuration tool:

Please Input The Administrator Password

Edit Data

Enter requested data.

Password is used for control popedom.

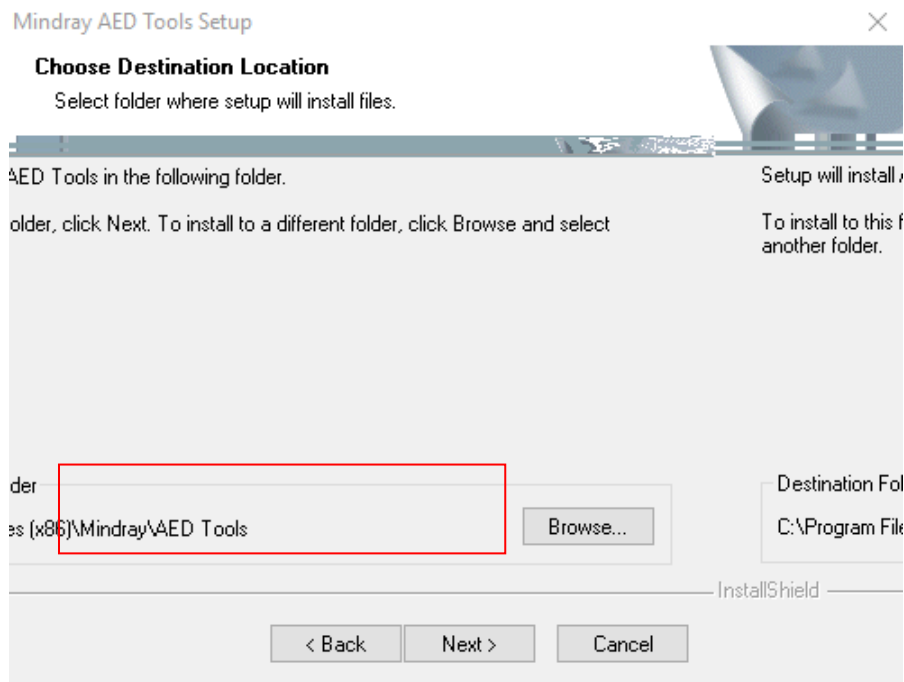
password

confirm

InstallShield

Next > Cancel < Back

6. Click **Next**, and select the installation directory.


7. Click **Next**, until the USB driver is installed successfully.
8. After the installation is complete, follow the prompts to choose whether you need to restart your computer now.



1.2.2 Updating the VCOM Driver

Note: The Windows 10 operating system can automatically recognize the VCOM driver. You may not need to perform VCOM driver operations. The Windows 7 operating system needs this operation.

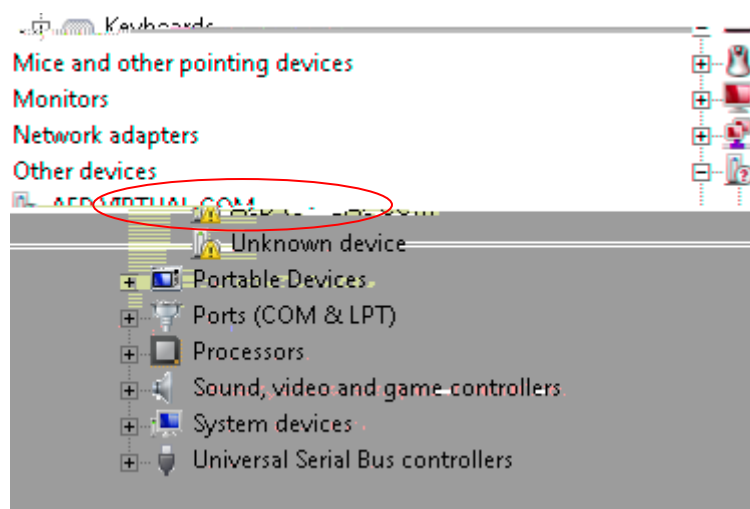
1. Open the AED lip to power it on. Use the USB cable to connect the PC and the USB micro port of the device. The device manager has the following device.

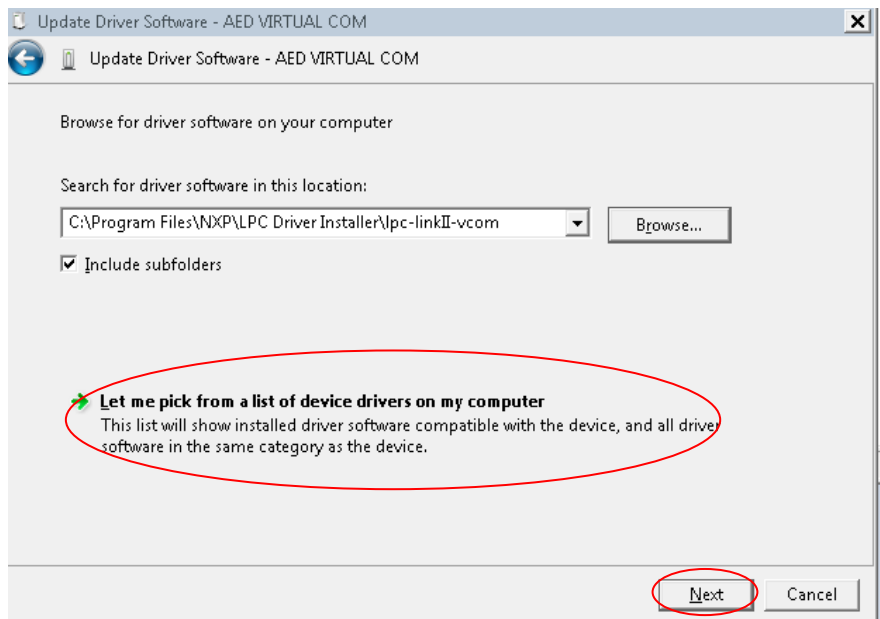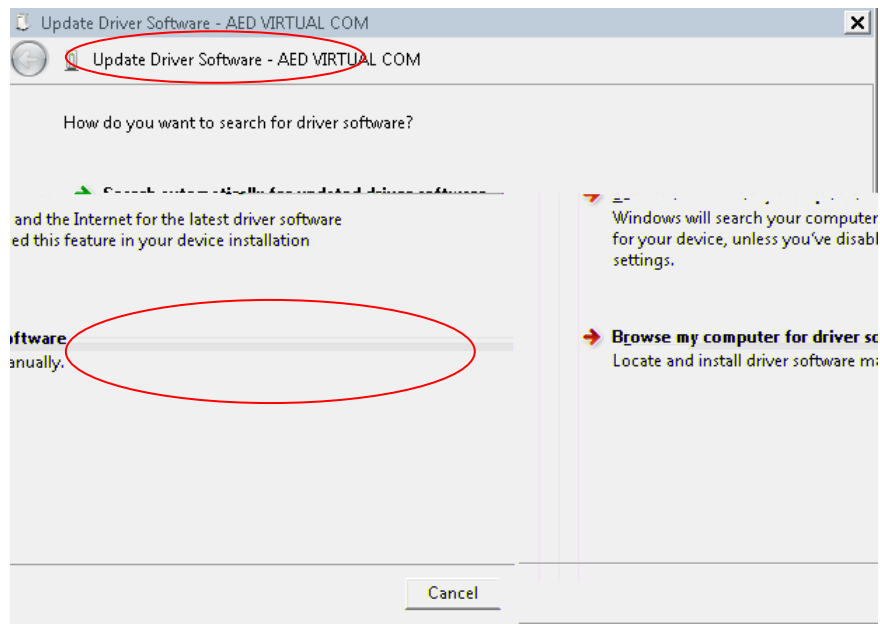
2. Right-click to select **Update Driver Software**.

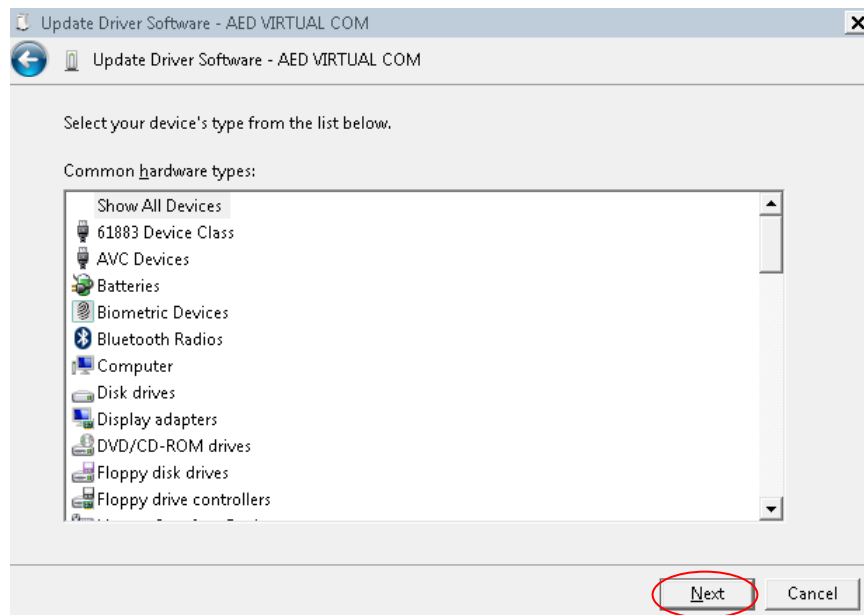
3. Directly select **Next**.

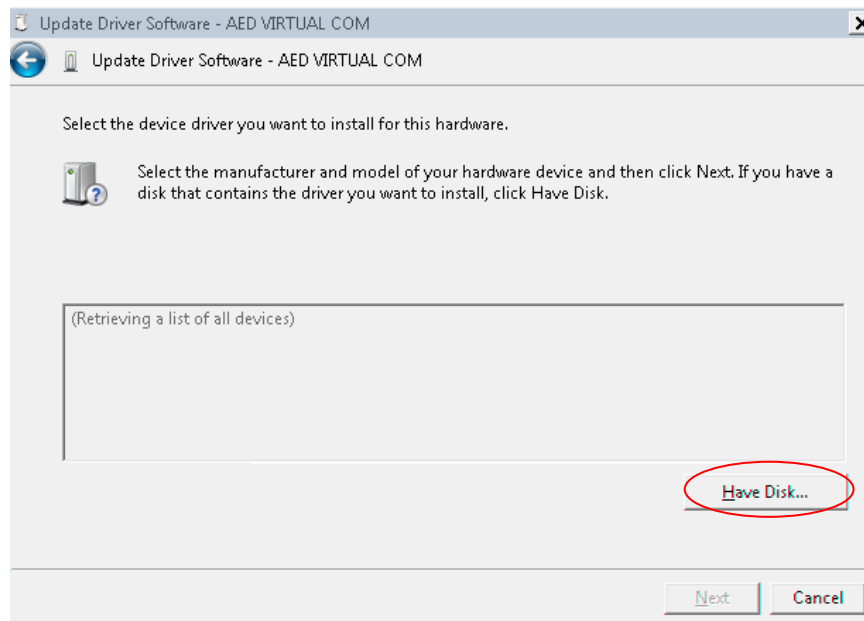
4. Select **Have Disk...**

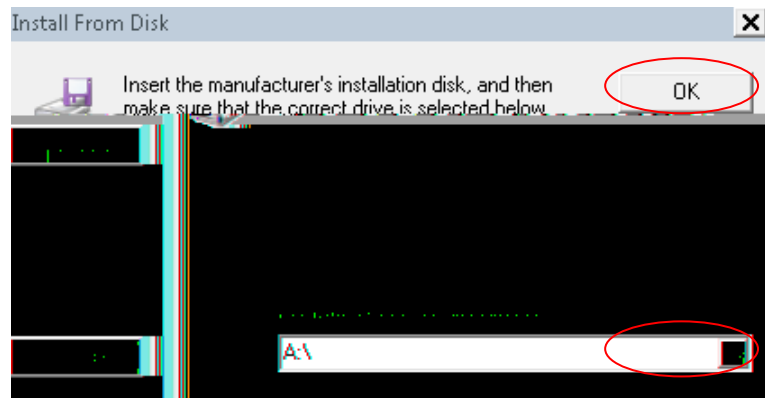
5. Browse to select the vcom installation directory.

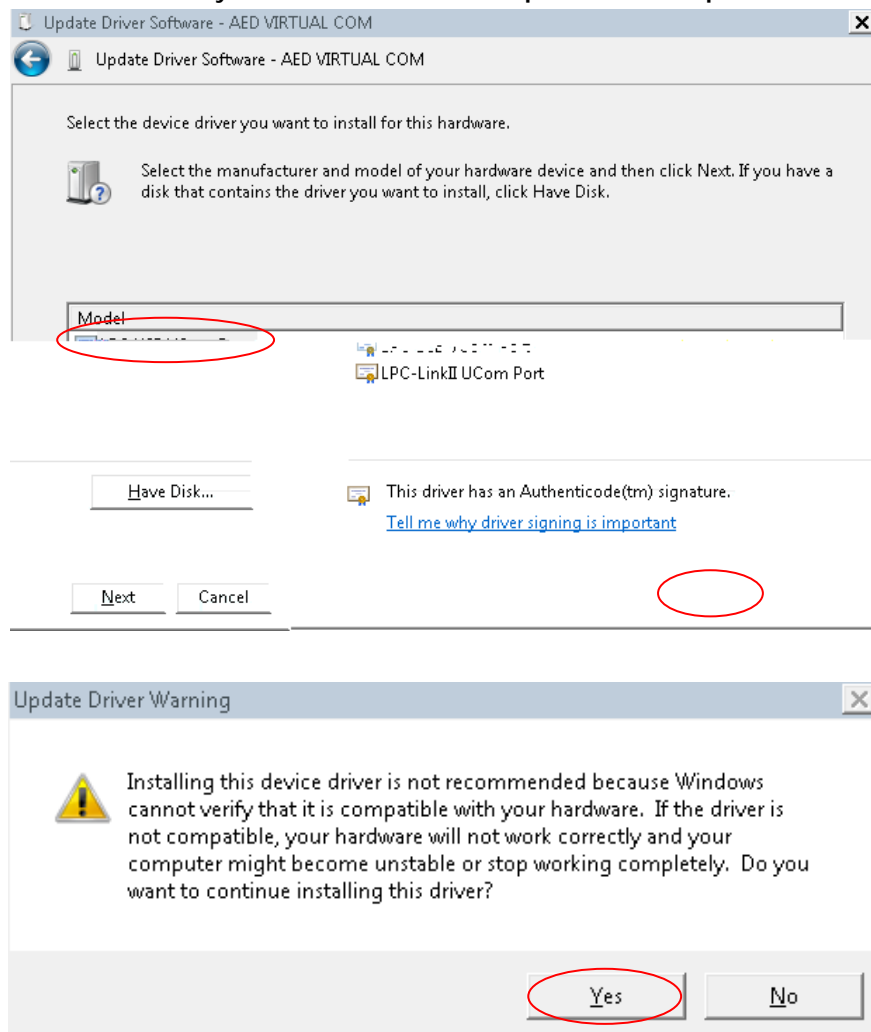
6. Select **Installation directory\NXP\LPC Driver Installer\lpc-linkII-vcom\lpc-vcom.inf**.

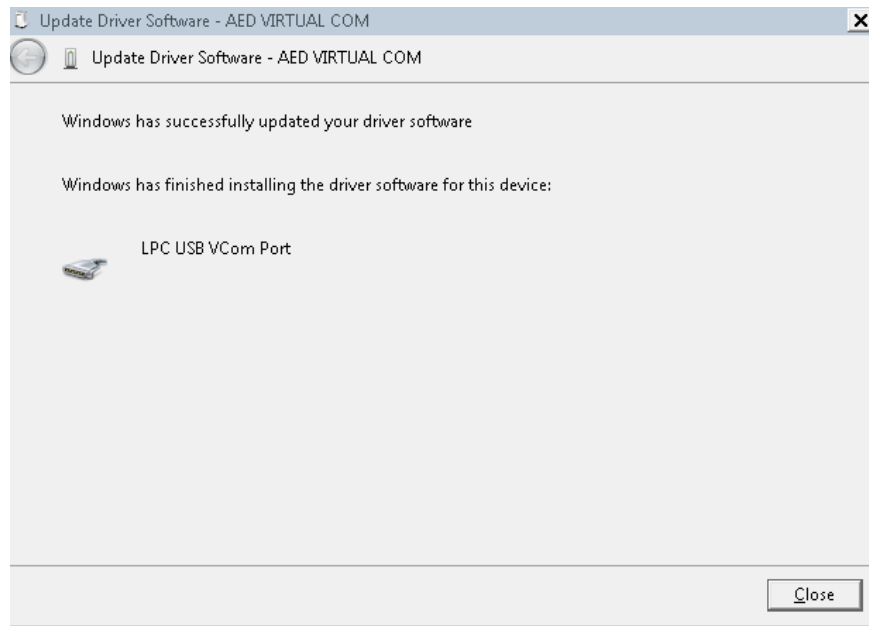7. Wait for updates, and close the window

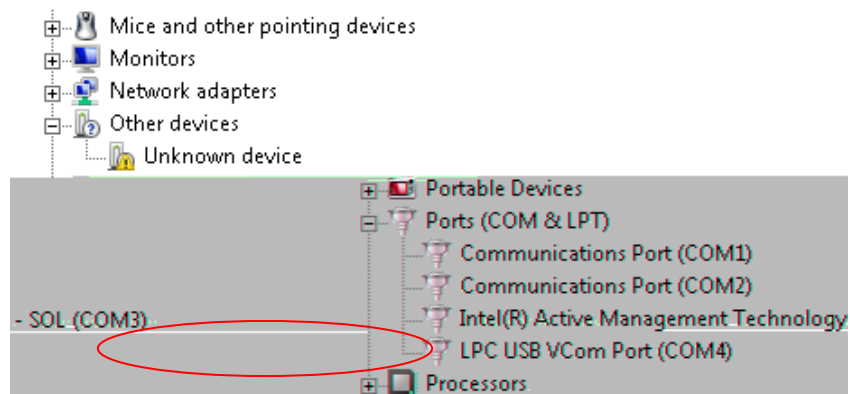8. After the completion, the device manager is as shown below, indicating that the VCOM recognition is successful.

1.2.3 Uninstalling the Software

1. Open the **Start** menu on the PC and enter the **Control Panel-> Programs and Functions** page. Select Mindray AED tools software, and click **Uninstall**.
2. An uninstallation prompt pops up. Select **Yes**, and follow the prompts to uninstall.

1.3 Software Operations

1.3.1 Connecting the Device

Power on the AED device, and use a USB cable to connect the AED device to the PC. After the connection, the AED Tool prompts that **“The AED device is detected, enter the configuration mode?”**. If you select **Yes**, the device enters the configuration mode and is connected with the AED Tool. If you select **No**, the device remains in non configuration mode, and is not connected with the AED Tool.

Note:

After you use the USB cable to connect the device with the AED Tool, you should wait for the prompt “The AED device is detected, enter the configuration mode?” about 30s to 60s. If this prompt is not displayed for a long time, you are recommended to re-plug the USB cable or replace the USB cable.

For the Windows 10 operating system, you should right click and operate the AED Tool as an administrator.

1.3.2 Identifying the Connection Status

Disconnected Device

Start the AED Tool software. If the

Connected Device:

Turn on the device and use the USB cable to connect it to the AED Tool. Select **Yes** to enter the configuration mode. There is a prompt **Device Connected** as shown below.

1.3.3 Managning Configurations

On the **Configuration** page, you can write the default configurations, open the configuration file, read the device

defiys15.82(d)-(d)21002(Tig)0.60.4T(w)004(052(w)3164 V6.20(9.21(r)-64876(O) e1870(e)273,20(d)91 Tw0/56002.Tv(16)ad(1)0g)-2..8n

Method 3 to write configurations:

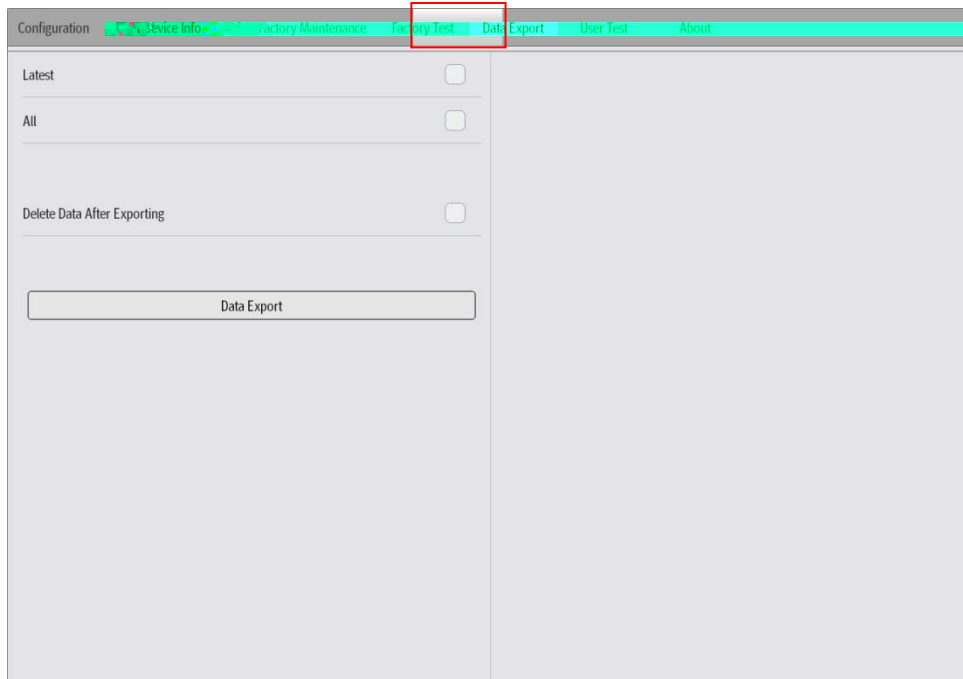
1. Enter the **Configuration** page, click **Write Default Configuration**, and the prompt "**Are you sure to restore factory settings?**" appears. Click **OK**, and the prompt "**Writing successfully! Effective after the device is restarted**" appears.
2. The **System Time**, **Language** and **WLAN Setup** are not changed after restoring the default configurations. You are recommended to confirm these configurations by clicking **Read Device Configuration** after restarting the device.

1.3.4

1.3.5 Exporting Data


Enter the **Data Export** page, select the latest file or all files, and click **Data Export** to export all selected data. Select **Delete Data After Exporting**, the data in the device is deleted after the data is exported completely.

Note: The exported data is stored in the \ImportData\Savior folder under the installation location directory of the AED Tool, and this folder is named with the serial number of the device.

1.3.6 Performing the User Test

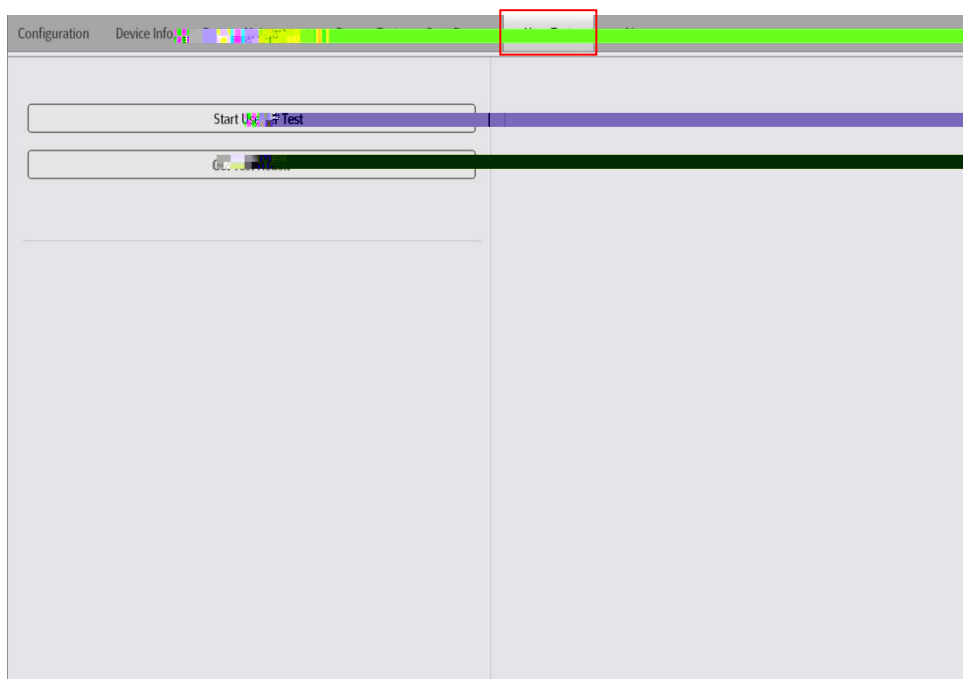With the device lip opened, enter the **User Test** page to perform the user test and obtain test results. Click **Start User Test**, the device starts the user test, and you should can perform the user test followed the voice and text prompts. After the test is completed, click **Get Test Result** to read the test results.


1.3.7 Performing the Network Test

Enter the **Configuration** page and find **WLAN Setup** by the up and down arrows.


For the device configured with the 4G module, after setting **AED Alert System Site** and **AED Alert System Port**,

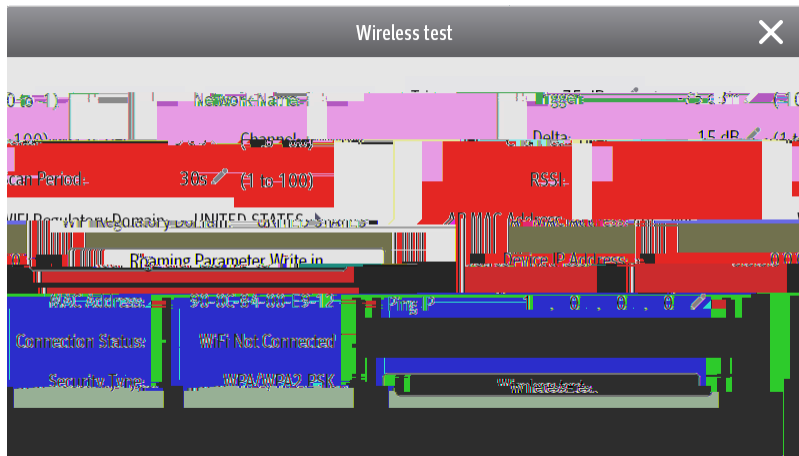click **Wireless Network Test** and wait for the wireless network to be completed.

1.3.8 Viewing the Software Version

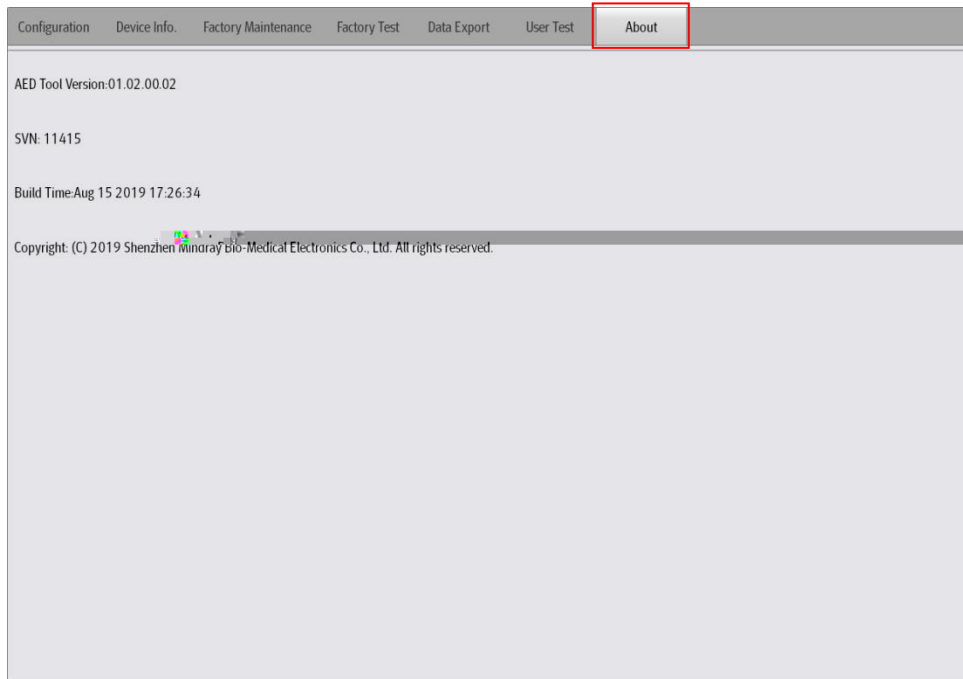Enter the **About** page, and view the detailed version information of the AED Tool.

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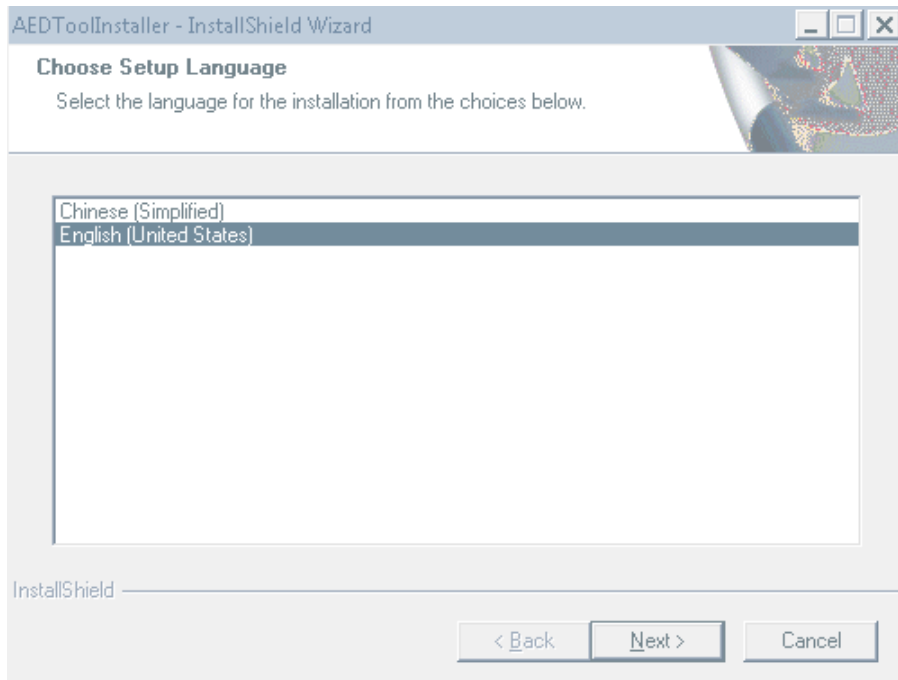
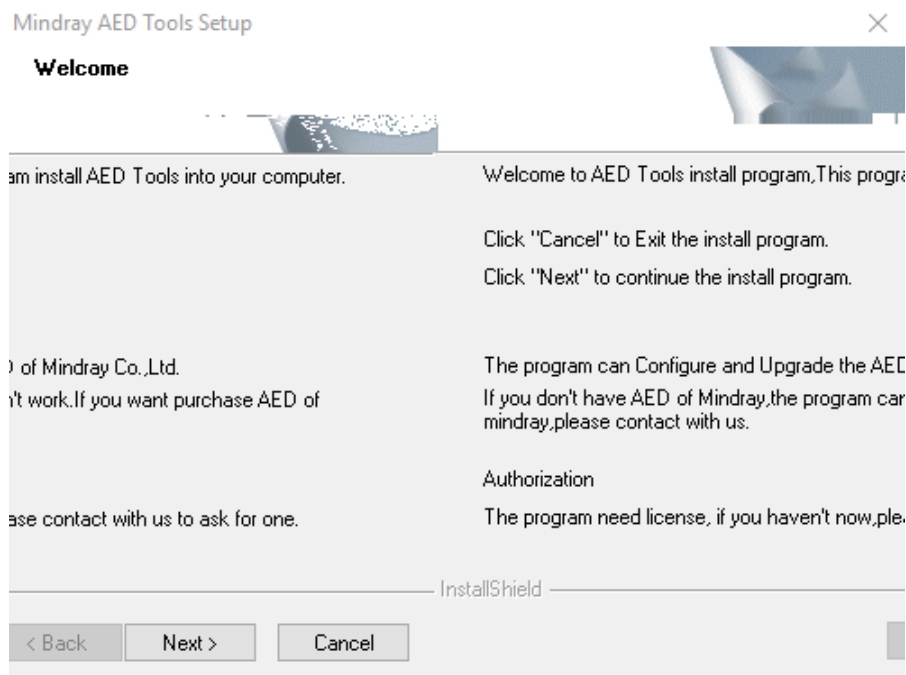
Mindray AED Tools Setup

Customer Information
Please enter your information.

Please enter your name, the name of the company for which you work and the product serial number.

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© Copyright 2019-2021 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. Все права защищены.