



BeneHeart C Series

Semi or Fully Automated External Defibrillator (AED)

Key Features



Open the lid to power on in 2 seconds



Choose between semi automatic or fully automatic models



Up to three languages built in. 27 languages to choose from



Pre-connected adult/child pads with 5 years lifecycle



Adult and child modes

Intelligent ResQNavTM

When using an AED in a public setting, most rescuers lack resuscitation experience and therefore need detailed guidance; however experienced rescuers may only need simple prompts to help them. Using comprehensive user research data, Mindray has developed ResQNavi technology which evaluates the AED user's proficiency and automatically provides targeted, intelligent rescue instructions based on their ability.

Adaptive rescue guidance

When the rescuer is inexperienced, the time it takes to apply the pads will be longer than if they were a proficient AED user. ResQNavi recognises this delay and provides more detailed guidance, walking the rescuer through the process with voice prompts.

The BeneHeart C2 AED also includes a 7" colour TFT screen, which provides animated guidance to the user, in addition to the audio commands.



Skilled rescuer

Remove clothing from patient's chest.

Apply pads as shown on pads.



Unskilled rescuer

Remove pads package from lid of AED. Tear open package. Apply pads as shown on pads.

Apply pads **firmly** to patient's **bare** chest as shown.


.....





Reliable and ready to respond


The Mindray BeneHeart C Series has been rigorously tested to ensure it provides ultimate reliability; so when called upon, no matter what the circumstance, it is ready to respond.



1.5m drop tested



IP55 water/dust proof



Meets all standards for use in
helicopters and other
emergency transport

The durable battery and pads have a
life cycle of 5 years from date of
manufacture

