



## AS3000 CLINICAL QUICK START GUIDE


### AS3000 Adult/Infant/Neonatal/Small Child

This guide describes the use of the AS3000 Adult/Infant/Neonatal/Small Child and Pediatric Monitor/ ventilator. Each device is a separate piece of equipment.

1. Ensure all flows are OFF and the Y-piece is OPEN prior turning ON the AS3000.
2. Follow the ventilator's onscreen instructions to complete the START UP process.
3. If the Leak Test fails, either use the manual mode, or connect a device to the Leak Test port.

**Start the ventilator by using the button** (highlighted in blue) on the MODE screen. Select CMV, PCV, PS, SIMV and the appropriate mode.

6. The **STANDBY** screen displays the current mode. See the manual for details on the MANUAL Mode of the ventilator.
7. When reducing N2O flow, O<sub>2</sub> decreases. Adjust O<sub>2</sub> flow as needed.
8. To turn OFF the volume, pressure and Apnea alarms in **MANUAL** mode, use the button on the ALARM Off screen.

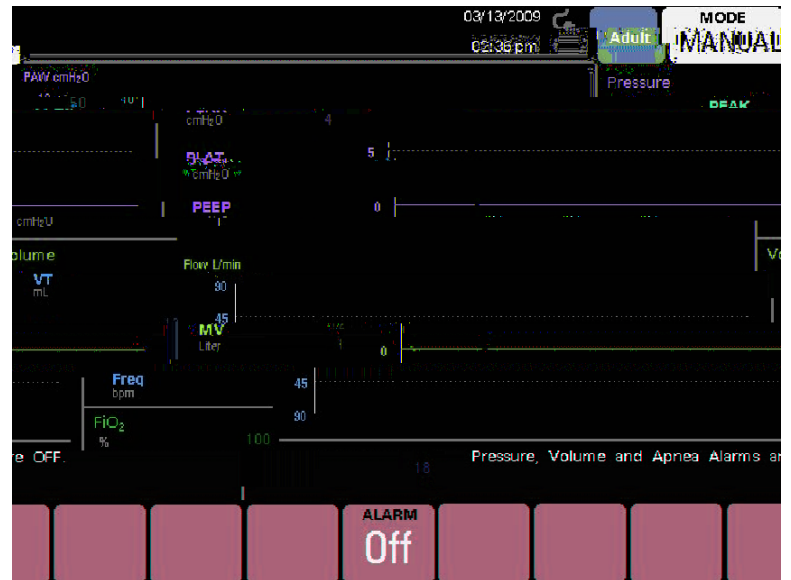

**Assistance is provided by the Mindray Technical Support Center from 9:00a to 5:00 EST.** Contact your local distributor for assistance. Please refer to the user manual for detailed information. Call 1-800-288-2121.



- 9. The MANUAL/AUTO key is checked back to default when the bag is disconnected.
- 10. Confirm the waste gas scavenger is connected (red fail indicator is on).
- 11. ADULT/CHILD size can be changed (i) 1 c 00 1 Tf 7124 02.0 0 2.0182.4 614.22

