



- 1 Touch screen
- 2 Patient
- 3 Probe
- 4 Review
- 5 Report
- 6 End Exam
- 7 Body Mark
- 8 Cine
- 9 Arrow
- 10 Clear
- 11 Comment
- 12 Station
- 13 Image Mode & B/2D Mode
- 14 Measure
- 15 Update
- 16 Save1
- 17 Zoom
- 18 Multi-functional knob
- 19 Touch
- 20 Depth

# DC-7

## Quick Reference Guide



**mindray**<sup>™</sup>  
NORTH AMERICA

## Getting Started

### New Patient

1. Press [Patient], fill in the basic patient information via Touch screen or keyboard.
2. Press [Probe], select probe and exam mode via Touch screen.

## Scanning

### B Mode

1. Adjust the interested parameters via Touch screen.
2. Rotate the [B] knob to adjust the gain.